

## **ABC Brand Latex Condoms With spermicidal lubricant**

If used properly, latex condoms will help to reduce the risk of transmission of HIV infection (AIDS) and many other sexually transmitted diseases, including chlamydia infections, genital herpes, genital warts, gonorrhea, hepatitis B, and syphilis.

This product combines a latex condom and a spermicidal lubricant. The spermicide, nonoxynol-9, reduces the number of active sperm, thereby decreasing the risk of pregnancy if some semen spills or seeps outside the condom. However, the extent of decreased risk has not been established. This condom should not be used as a substitute for the combined use of a vaginal spermicide and a condom.

### **Precautions:**

- Use a new condom every time you have sexual intercourse or other acts between partners that involve contact with the penis.
- Do Not Reuse Condoms.
- Store condoms in a cool, dry place.
- If the rubber material is sticky or brittle or obviously damaged, do not use it.
- If a lubricant is wanted, use water-based lubricants such as K-Y Jelly. *DO NOT USE OIL-BASED LUBRICANTS*, such as those made with petroleum jelly (e.g., Vaseline®), mineral oil, vegetable oil, or cold cream, as these may damage the condom.

### **Directions for Use:**

- Put the condom on after the penis is fully erect and *before* intimate contact. Lesions, preejaculate secretions, semen, vaginal secretions, saliva, urine, and feces can all transmit disease organisms.
- Place the condom on the head of the penis and unroll or pull it all the way to the base.
- Leave an empty space at the end of the condom to collect semen. Remove any air remaining in the tip of the condom by gently pressing the air out toward the base of the penis.
- After ejaculation and while the penis is still erect, hold onto the rim of the condom so that the condom does not slip off as the penis is carefully withdrawn.

**CAUTION: This Product Contains Natural Rubber Latex  
Which May Cause Allergic Reactions.**