

Veterinary Services

Beef 2007-08 Ear Notch Sample Collection Record

National Animal Health Monitoring System

2150 Centre Ave, Bldg B Fort Collins, CO 80526

Form Approved
OMB Number 0579-NEW

Complete the following information:

State:	Producer self-selected ID:		Date samples shipped:
Normhau af bayea abi	nuina to dove	Niverban of	(mm/dd/yy)
Number of boxes shipping today:		Number of	samples in this shipment:

Quick Reference:

Detailed collection instructions are attached.

- 1. Complete the above information for each set of samples sent to the lab.
- 2. Collect an ear notch from each calf born between November 1, 2007, and July 15, 2008. Place each sample in a separate whirlpak bag, express air, and twist down twice.
- 3. IT IS CRITICAL that all data on the label are completed with a permanent pen (Sharpie®) and affixed correctly. Getting the correct result back for each calf will be important for future management decisions.
- 4. Freeze the samples as soon as possible and for at least 24 hours. Freeze icepacks for at least 24 hours before shipping.
- To ship, place samples in the larger bag and secure with rubber band, twist tie, or knot.
 Place 2 ice packs in the bottom of the supplied shipping box, the samples, and then 2 more ice packs.
- 6. Place the YELLOW copy of this submission form between the inner and outside flaps of the box. Mail the ORIGINAL to the NAHMS program using the supplied Business Reply envelope. Keep the PINK copy for your records.
- 7. Contact FedEx for the nearest location that accepts EXPRESS packages and note the last drop-off time. Ship on Monday, Tuesday, or Wednesday.