

SALIVA COLLECTION INSTRUCTIONS - MOTHER

Step 1: Record time		Step 2: Select tube	
	Record time on sample collection sheet		Match tube color with label on collection sheet
Step 3: Mother saliva collection method) (straw OR swab			
	<p><u>Straw method</u></p> <ol style="list-style-type: none"> 1) Insert straw in mouth 2) Allow saliva to collect in tube (1mL) 3) Cap the tube, discard straw 		<p><u>Swab method</u></p> <ol style="list-style-type: none"> 1) Insert swab under tongue, close mouth 2) Hold for 2 minutes until saturated 3) Place swab in tube and cap the tube
Step 4: Freeze sample		Good Job!	
	Immediately place sample in freezer		

Public reporting burden for this collection of information is estimated to average 5 minutes per response, including the time for reviewing instructions, searching existing data sources, gathering and maintaining the data needed, and completing and reviewing the collection of information. **An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB control number.** Send comments regarding this burden estimate or any other aspect of this collection of information, including suggestions for reducing this burden, to: NIH, Project Clearance Branch, 6705 Rockledge Drive, MSC 7974, Bethesda, MD 20892-7974, ATTN: PRA (0925-0647*). Do not return the completed form to this address.

SALIVA COLLECTION INSTRUCTIONS – INFANT/CHILD

<h2>Step 1: Record time</h2>		<h2>Step 2: Select tube</h2>	
	<p>Record time on sample collection sheet</p>		<p>Match tube color with label on collection sheet</p>
<h2>Step 4: Infant/Child saliva collection</h2>			
	<ol style="list-style-type: none"> 1) Firmly grasp one end of swab with gloved hand 2) Place other end of swab under child's tongue for 2 minutes <p>Do not let go of swab!</p>		<ol style="list-style-type: none"> 3) Switch sides of swab and repeat process 4) Fold swab 5) Place folded swab in tube and cap the tube 6) Discard glove
<h2>Step 6: Freeze sample</h2>		<h2 style="color: red;">Good Job!</h2>	
	<p>Immediately place sample in freezer</p>		