


Form Approved
OMB No. 0920-New
Expiration Date: XX/XX/XXXX

**Understanding HIV/STD Risk and Enhancing PrEP Implementation Messaging in a
Diverse Community-Based Sample of Gay, Bisexual, and Other Men Who Have
Sex with Men in a Transformational Era (MIC-DROP)**


**Attachment 4f
STI Sample Collection English**

Public reporting burden of this collection of information is estimated to average 30 minutes per response, including the time for reviewing instructions, searching existing data sources, gathering and maintaining the data needed, and completing and reviewing the collection of information. An agency may not conduct or sponsor, and a person is not required to respond to a collection of information unless it displays a currently valid OMB control number. Send comments regarding this burden estimate or any other aspect of this collection of information, including suggestions for reducing this burden to CDC/ATSDR Reports Clearance Officer; 1600 Clifton Road NE, MS D-74, Atlanta, Georgia 30333; Attn: OMB-PRA (0920-New)

This Kit Includes:

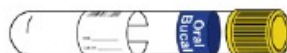

1 collection tube

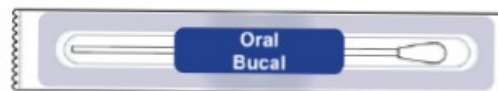

1 oral swab




1 specimen bag
(with absorbent pad)


READ FIRST

1. Read all instructions thoroughly before collecting your specimens.
2. Do not eat, drink, smoke, or brush your teeth for 30 minutes prior to collection of the oral swab specimen.
3. Do not drink alcohol on the day of collection.
4. Collection tubes may contain a liquid preservative. **DO NOT EMPTY LIQUID PRESERVATIVE FROM THE COLLECTION TUBES IF PRESENT.**

TO PREVENT REJECTED SPECIMENS, PLEASE READ ALL INSTRUCTIONS BEFORE BEGINNING COLLECTION

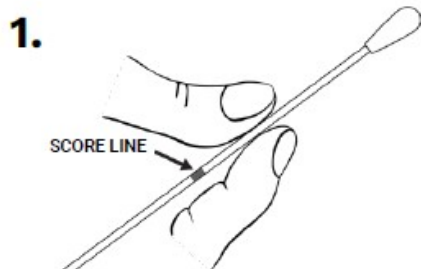


1. Hold the swab between the faint score line (if present) and the cotton-tipped portion of the swab.

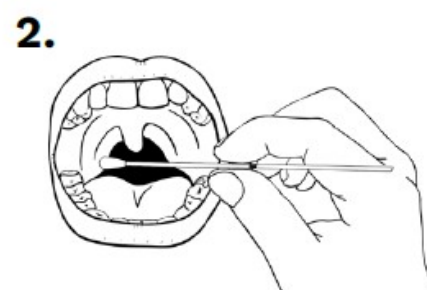


2. Insert the swab into your mouth and rub swab tip against the back of your throat for 10 seconds.




3. Carefully withdraw the swab and place into the collection tube marked with a **BLUE "ORAL" LABEL**. Break the swab by bending against the collection tube.

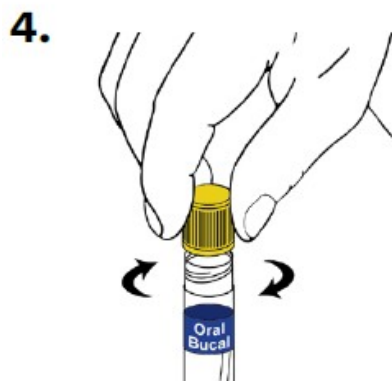

4. Place lid onto the collection tube and ensure it is closed evenly and tightly.

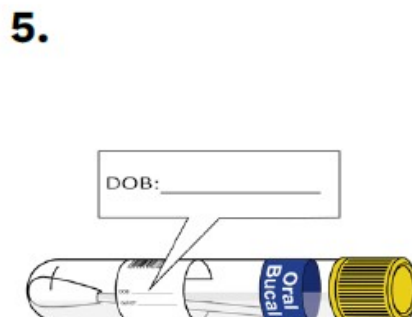

5. Write your date of birth in MM/DD/YYYY format onto the tube in the designated area (DOB).

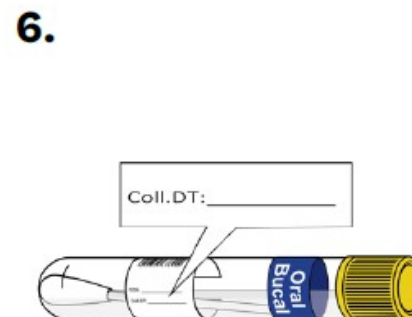

6. Write the specimen collection date (today's date) in MM/DD/YYYY format onto the tube in the designated area (Coll. Date).



7. Place specimen into the empty biohazard bag with the absorbent pad and ensure the seal is closed completely.

This Kit Includes:



1 collection tube



1 pipette



1 collection cup

OR



1 pee canter



1 specimen bag
(with absorbent pad)

READ FIRST

1. Read all instructions thoroughly before collecting your specimens.
2. Do not drink alcohol on the day of collection.
3. Do not collect specimens during menstruation or within 24 hours of intercourse.
4. Collection tubes may contain a liquid preservative. **DO NOT EMPTY LIQUID PRESERVATIVE FROM THE COLLECTION TUBES IF PRESENT.**


TO PREVENT REJECTED SPECIMENS, PLEASE READ ALL INSTRUCTIONS BEFORE BEGINNING COLLECTION

1.




Do not urinate for at least 1 hour prior to collection. Using first morning urine is best.

2.

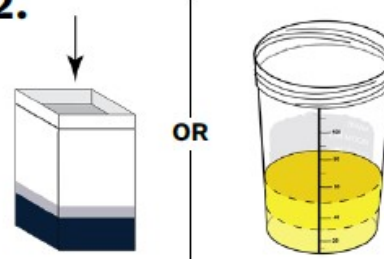


Urinate directly into the pee canter or collection cup, filling between 1/3 and 1/2.

3.

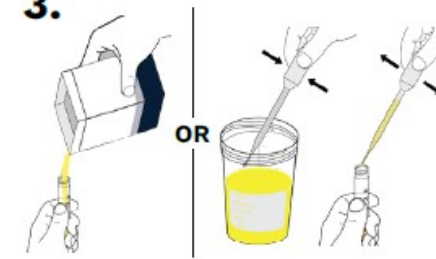


Pour from the pee canter or use the pipette to transfer urine from the collection cup into the collection tube(s) marked with an **ORANGE "URINE" LABEL**.

4.

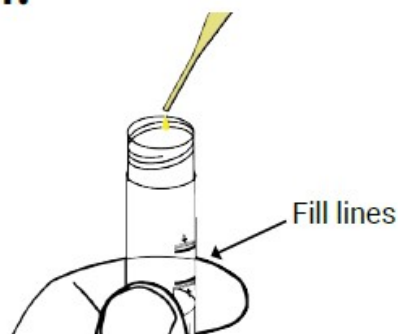


Fill the collection tube(s) until the combined liquid is between the designated fill lines. **Do not fill tube past the maximum fill line or specimen will be rejected.**

5.

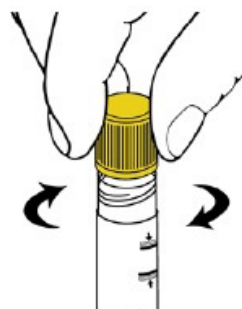

Place lid onto the collection tube and ensure it is closed evenly and tightly.

6.

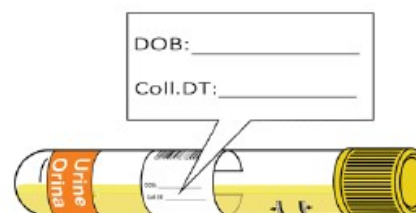


Write your date of birth and the specimen collection date (today's date) in MM/DD/YYYY format onto the tube in the designated areas.

7.



Place specimen into an empty biohazard bag with the absorbent pad and ensure the seal is closed completely.

This Kit Includes:

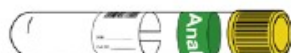

1 collection tube



1 anal swab




1 specimen bag
(with absorbent pad)

READ FIRST

1. Read all instructions thoroughly before collecting your specimens.
2. Do not drink alcohol on the day of collection.
3. Collection tubes may contain a liquid preservative. **DO NOT EMPTY LIQUID PRESERVATIVE FROM THE COLLECTION TUBES IF PRESENT.**


TO PREVENT REJECTED SPECIMENS, PLEASE READ ALL INSTRUCTIONS BEFORE BEGINNING COLLECTION

1.

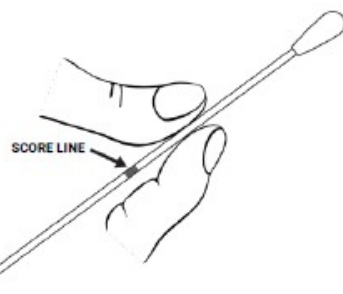

Hold the swab between the faint score line (if present) and the cotton-tipped portion of the swab.

2.

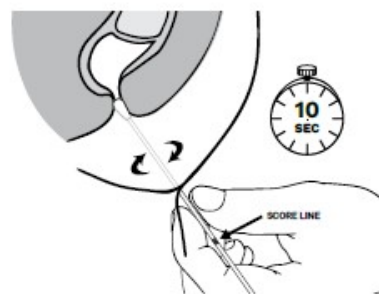


Insert the swab 3 – 5cm (1 – 2") into the anal canal. Gently turn the swab for 5 – 10 seconds to collect any potential organisms.

3.



Carefully withdraw the swab and place into the collection tube marked with a **GREEN "ANAL" LABEL**. Break the swab by bending against the collection tube.

4.

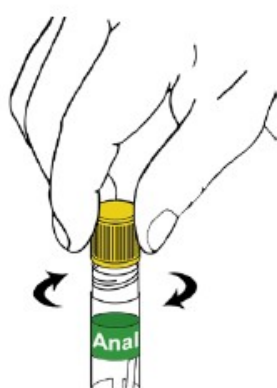

Place lid onto the collection tube and ensure it is closed evenly and tightly.

5.



Write your date of birth in MM/DD/YYYY format onto the tube in the designated area (DOB).

6.



Write the specimen collection date (today's date) in MM/DD/YYYY format onto the tube in the designated area (Coll. Date).

7.



Place specimen into the empty biohazard bag with the absorbent pad and ensure the seal is closed completely.